

SeaBrook Cementation Sheet

Resin Cements for use with Empress and e.max			
Brand	Manufacturer	Empress	e.max
RelyX ARC	3M ESPE	Yes	Yes
RelyX Veneer	3M ESPE	Yes	Yes
C&B Cement	Bisco	Yes	Yes
Choice	Bisco	Yes	Yes
Illusion	Bisco	Yes	Yes
Insure	Cosmedent	Yes	Yes
Prevue	Cosmedent	Yes	Yes
Ultra-Bond Quik	Denmat	Yes	Yes
Calibra	Dentsply	Yes	Yes
Appeal	Ivoclar Vivadent	Yes	Yes
Multilink	Ivoclar Vivadent	Yes	Yes
Variolink II	Ivoclar Vivadent	Yes	Yes
Nexus	Kerr	Yes	Yes
Panavia 21	Kuraray	Yes	Yes
Panavia F	Kuraray	Yes	Yes
C&B Metabond	Parkell	Yes	Yes
Cement-it	Pentron	Yes	Yes
Lute-it	Pentron	Yes	Yes
Conventional Cements for use with Empress and e.max			
VivaGlass CEM	Ivoclar Vivadent	No	Yes
Fuji 1	GC	No	Yes
Ketac CEM	3M ESPE	No	Yes
RelyX Unicem	3M ESPE	No	Yes
RelyX Luting Cement	3M ESPE	No	Yes
Fuji CEM	GC	No	Yes
Fuji Plus	GC	No	Yes
Material	IPS e.max Press Lithium disilicate		
Indication	Veneers ²⁾ , partial crowns ²⁾ , anterior and posterior crowns 3-unit bridges up to the 2nd premolar		
Cementation method	Adhesive cementation	Self-adhesive / conventional cementation	
	Yes	Yes	
Etching	20 sec. with IPS Ceramic Etching Gel		
Conditioning / Silanization	60 sec. with Monobond-S	1)	
Cementation system	Variolink® Veneer Variolink® II Multilink® Automix	Vivaglass® CEM	

¹⁾ For adhesive cementation, the restorations must be silanized.

²⁾ Partial Crowns and veneers must be adhesively cemented.